




DAILY CARE & CLEANING PROTOCOL

FLOCARE® BUTTON EXTENSION SET



Minimize product handling and manual handling of the product, specifically the connectors, by hand as to reduce the risk of any hand touch contamination. FloCare® Button Extension Sets are disposables and should be replaced at least every 2 weeks. Wash the FloCare® Button Extension Set after every use thoroughly using the following instructions as to reduce the risk of contamination of the device which might cause patient infection or cross-infection. Contamination of the device may lead to injury, illness or death of the patient.

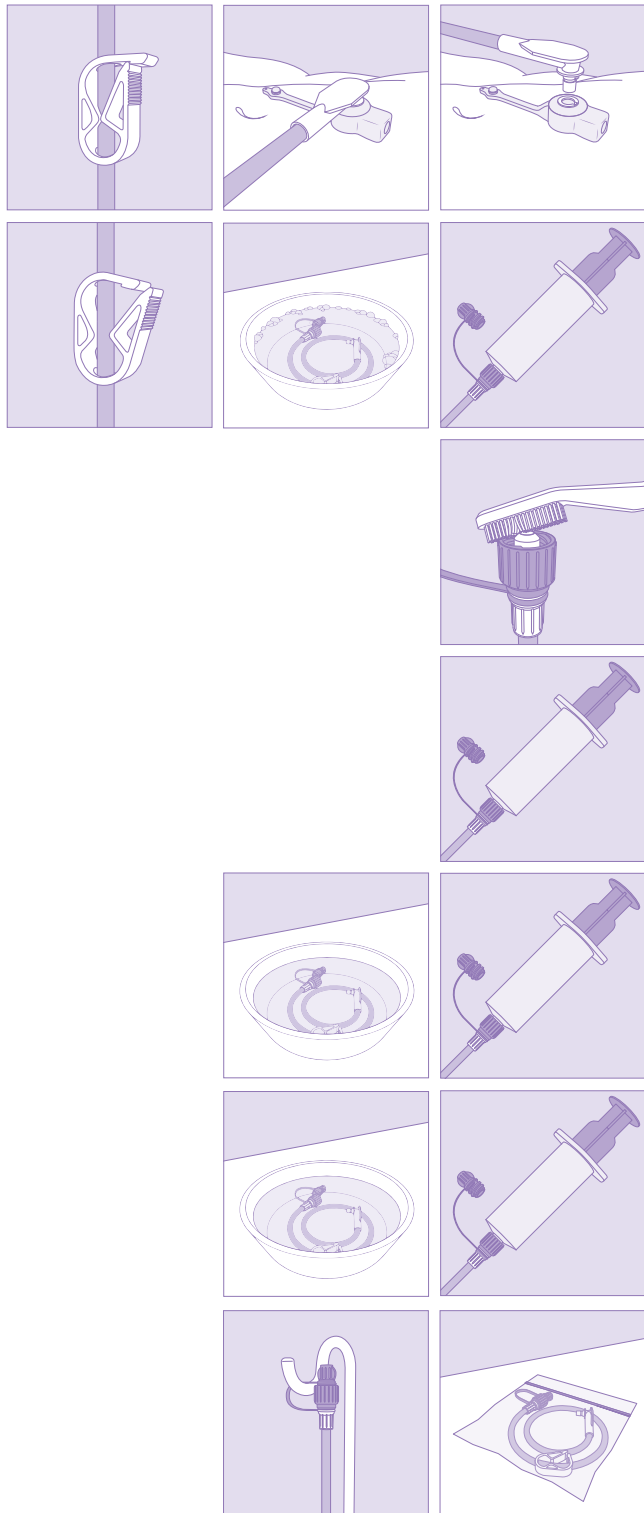

1 Close the clamp and turn the extension set anti-clockwise to match up the markings. Gently pull the extensions set to remove from the button.

2 Open the clamp and wash thoroughly in warm soapy water and rinse with at least 60 ml applied with a syringe.


Clean connectors using any brush as to remove any nutrition build up in the connector threads.

3 Flush with at least 60 ml sterile /cool boiled water as to remove any soap remnants.

4 Flush tube with at least 60 ml cold water sterilizer (up to 2% sodium hypochlorite and 16.5% sodium chloride) / bleach solution suitable to use with plastic products.

5 Rinse thoroughly with sterile / cool boiled water as to remove any sterilizer agent.

6 Allow to air dry, preferably in a hanging position. Store in air tight container or zip lock bag until next use.

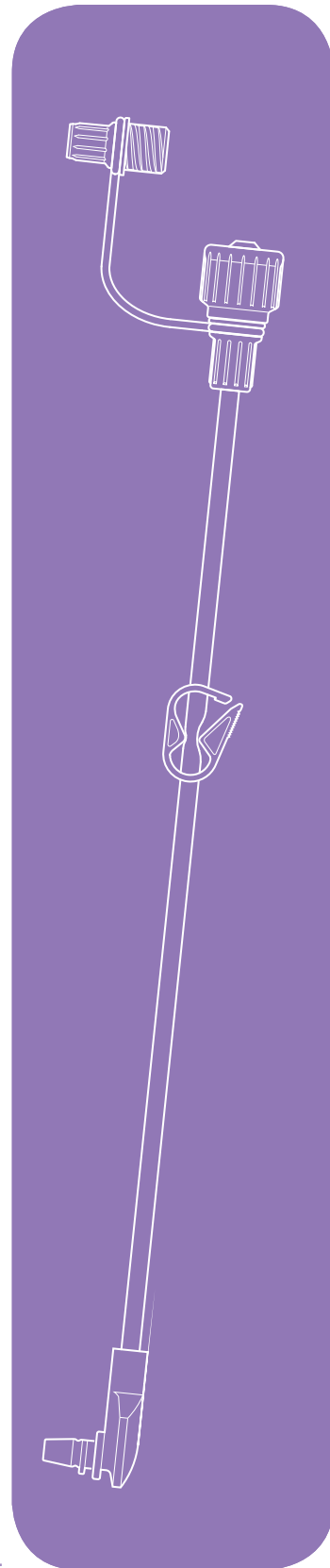

The extension set may be used for up to two consecutive weeks, if the connectors and their bonding to the tubing are still intact. Total use time can vary, and replacement of the product is based on medical scrutiny and viability of the product. In case any cracking, dislocation or discoloring is observed, a new extension set is to be used. After use, dispose the product and packaging in accordance with hospital, administrative and/or local government policy.

Manufacturer: Nutricia Medical Devices BV, Taurusavenue 167, 2132 LS Hoofddorp, The Netherlands
Production location: Nutricia Pharmaceutical Wuxi Co., Ltd., No. 17 Xinming Road, Wuxi High-tech Development Zone, P.R. China