NUTRICIA  
flocare®

## FLOCARE® PEG SET

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**UK-IE-AU-NZ • PEG SET** This is a medical device. For enteral use only / Contains loose parts which may be dangerous to young children / Non pyrogenic / DEHP-free / Latex-free / ARTG# 120000.

September 2019 - rev 200343\_C

**M** Manufacturer: Nutricia Medical Devices BV, Taurusavenue 167, 2132 LS Hoofddorp, The Netherlands

**P** Production location: Nutricia Pharmaceutical Wuxi Co., Ltd., No. 17 Xinming Road, Wuxi High-tech Development Zone, P.R. China

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
PERKÜTAN ENDOSKOPIK GASTROSTOMİ SETİ

### UK-IE-AU-NZ • INSTRUCTIONS FOR USE

PERCUTANEOUS ENDOSCOPIC GASTROSTOMY SET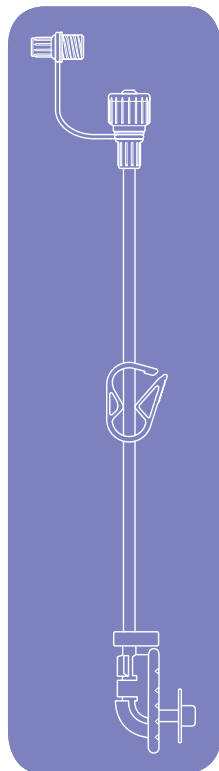

**Manufacturer:** Nutricia Medical Devices BV, Taurusavenue 167, 2132 LS Hoofddorp, The Netherlands

**Production location:** Nutricia Pharmaceutical Wuxi Co., Ltd., No. 17 Xinming Road, Wuxi High-tech Development Zone, P.R. China

STERILE EO


ENTERAL ONLY


200343\_C

outer cover back page (2)

outer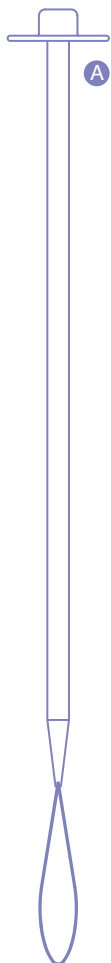


A

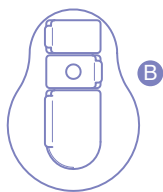
B


C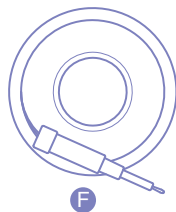


F

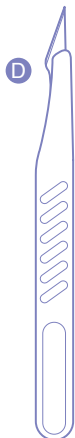D

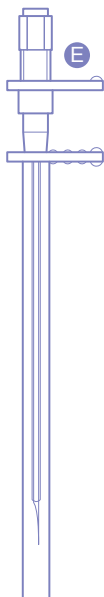E


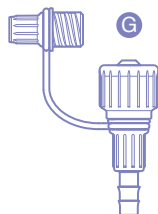
G

H

outer cover right page to be folded inside (3)

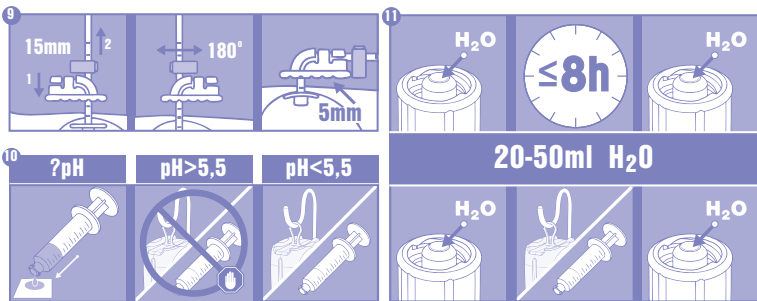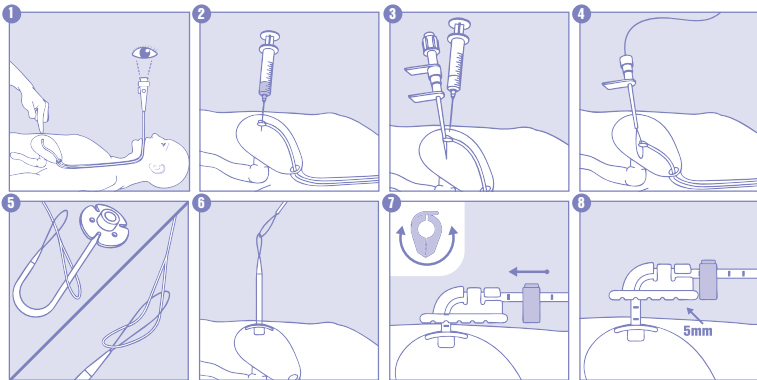<b>BE-NL</b>	N.V. Nutricia België, Werkhuizenkaai 160 - 1000 Brussel, België. Nutricia Nederland B.V., Antwoordnummer 10230, 2700 VB Zoetermeer-NL, Nutricia Medische Voedingsservice: 0800-022 33 22 (gratis).
<b>DE-CH-AT-LU</b>	Nutricia GmbH, D-91015 ERLANGEN, Deutschland Nutricia GmbH, A-1120 WIEN, Österreich Nutricia SA, CH-8005 ZÜRICH, Schweiz Produkt- und Ernährungsberatung D-A-CH 00800 - 700 500 00
<b>DK</b>	Nutricia A/S, Rørmosevej 2A, 3450 ALLERØD
<b>ES</b>	Nutricia SRL, C/ Torrelaguna 77, 6ª plta. CP 28043, Madrid, España. Teléfono de atención al cliente: (+34) 900 211 088. <a href="http://www.nutricia.es">www.nutricia.es</a> .
<b>FR-BE-CH-LU</b>	NUTRICIA Nutrition Clinique, 150 Bd Victor Hugo, 93406 Saint-Ouen Cedex, France. <a href="http://www.nutricia.fr">www.nutricia.fr</a> N.V. Nutricia België, Quai des Usines, 160 - 1000 Bruxelles, Belgique. Nutricia S.A. Suisse, CH-8005 ZÜRICH
<b>FI</b>	Nutricia Medical Oy, Linnankatu 26 A, 20100 TURKU
<b>GR</b>	Numil Hellas A.E. 17 χλμ. Αθηνών Λαμίας & Καλαμάτας 2, 14564 Κηφισιά Γραμμή φροντίδας: 800 11 68600, 210 6248 547
<b>IT-CH</b>	Nutricia Italia SpA, via C. Farini 41 - 20159 Milano - Numero Verde 800822096 Nutricia SA Svizzera, CH-8005 ZÜRICH
<b>NO</b>	Nutricia Norge AS, Drammensveien 123, 0277 OSLO
<b>PT</b>	Distribuído em Portugal por: Nutricia Advanced Medical Nutrition – Unipessoal, Lda. Avenida D. João II nº41 3º Piso, Torre Zen, 1990-084 LISBOA
<b>SE</b>	Nutricia, Box 3142, 169 03 SOLNA
<b>CZ</b>	Kontakt pro ČR: Nutricia a.s., V parku 2294/2, 148 00 Praha 4 - Chodov, infolinka: 800 110 001.
<b>EE</b>	NutriMedical OÜ, Tallinn 11313, Eesti
<b>HU</b>	Forgalmazza: NUMIL Kft., 1134 Budapest, Váci út 35. <a href="http://www.nutricia.hu">www.nutricia.hu</a>
<b>LT</b>	NUTRICIA SIA, Gustava Zemgala gatve 74, Ryga, LV-1039, Latvija
<b>LV</b>	NUTRICIA SIA, Gustava Zemgala gatve 74, Rīga, LV-1039, Latvija
<b>PL</b>	Nutricia Polska Sp. z o.o., ul. Bobrowiecka 8, 00-728 WARSZAWA
<b>SK</b>	Kontakt pre SR: Nutricia s.r.o., Prievozká 4, 821 09 BRATISLAVA, SR. Infolinka: 0800 444 006
<b>TR</b>	Numil Gıda Ürünleri San. ve Tic. A.Ş. Yapı Kredi Plaza B Blok No:1B Kat:8-9 Cömert Sok. Levent Mah. Beşiktaş-İSTANBUL
<b>UK-IE</b>	Nutricia Ltd, White Horse Business Park, TROWBRIDGE, Wiltshire, BA14 0XQ, England. Tel: 01225 711 688, <a href="http://www.nutricia.co.uk">www.nutricia.co.uk</a> Patient/Carer Information Number (UK only) Tel: 08457 623 653. Nutricia Ireland Ltd., Block 1, Deansgrange Business Park, Deansgrange, Co. DUBLIN, Ireland. <a href="http://www.nutricia.ie">www.nutricia.ie</a>
<b>AU-NZ</b>	Nutricia Australia Pty Ltd, Lvl 4, Bld D, 12-24 Talavera Rd, Macquarie Park NSW 2113, Australia, 1800 060 051, <a href="http://www.nutriciamedical.com.au">www.nutriciamedical.com.au</a> Nutricia Ltd, 124 Favona Rd, Favona, Auckland 2024, New Zealand, 0800 636 228, <a href="http://www.nutriciamedical.co.nz">www.nutriciamedical.co.nz</a> .

[www.nutriciaflocare.com](http://www.nutriciaflocare.com)

inner cover middle page (5) (opposite of outer front page 2)

inner

inner cover page right (6) - opposite of back page 1.

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## UK-IE-AU-NZ • PEG SET

The Flocare® Percutaneous Endoscopic Gastrostomy set (PEG) is intended to be placed using the "pull" technique, under endoscopic control.

Based on ESPEN Guidelines on enteral nutrition – Percutaneous endoscopic gastrostomy (PEG). Clinical Nutrition (2005)24, 848-861.

### CONTENTS

- A clear polyurethane tube (A), 40 cm in length, with
  - A radio-opaque line
  - A three-winged silicone internal retention disc
  - A Hydromer® coated tip with loop, for easy passage through the abdominal wall
  - Cm markings up to 12 cm from the internal disc
- An external disc, made of silicone, for fixation and prevention of kinks in the tube, for maximum patient comfort (B)
- Blue safety clamp for required tube fixation (C)
- A scalpel (D)
- An introducer needle with a trocar (E)
- A wire with loop, for a convenient and secure connection with the tube loop (F)
- A ENFit feeding connector (G)
- A quick release clamp (H).

The Flocare® PEG is available in CH 10, CH 14 and CH 18, all are 40 cm in length.

### INDICATIONS

- Long-term tube feeding via a gastrostomy.
- Stomach decompression.

The PEG may avoid possible inconveniences related to the long term application of nasogastric tubes and can aid in improving patients' mobility.

Unless contraindications are present, PEG placement is applicable for patients with a wide variety of diseases. A PEG is often used for neurologic patients, disabled or mentally retarded children, chronically ill and malnourished patients.

## CONTRAINDICATIONS FOR PEG PLACEMENT

- Severe coagulation disorders (e.g. haemophilia)
- Severe ascites and severe gastritis
- Ileus
- Anorexia nervosa and severe psychoses
- Acute pancreatitis and acute peritonitis
- When it is impossible to bring the anterior part of the stomach against the abdominal wall (subtotal gastrectomy, ascites, hepatomegaly, etc.)
- Marked peritoneal carcinomatosis
- Extensive tumor infiltration in the area of the puncture site

It will be very difficult and dangerous to place a PEG in case of oesophageal stenoses.

Lack of diaphanoscopy should carefully be taken into consideration. It is recommended to perform a diaphanoscopy. However, a negative needle aspiration test can be sufficient. (ESPEN, 2005)

## INSTRUCTIONS FOR USE

The PEG insertion is performed with the patient in a supine position. The patient is prepared as usual for endoscopy. Thorough oral and pharyngeal hygiene is very important. Administration of a prophylactic antibiotic should be considered. If possible, explain the procedure to the patient.

In addition to the Flocare® PEG set, the following equipment is required:

- Flexible endoscope with biopsy forceps
  - Sterile gauze
  - Skin disinfectant
  - Syringe with local anaesthetic and long IM needle
1. Normally PEG placement is executed under mild sedation. If this is not the case, a topical anaesthetic of the vocal cords has to be considered. Pass the endoscope into the stomach and inflate the stomach. With the room lights dimmed, locate the appropriate puncture site in the left upper quadrant of the stomach, by placing a finger at the point of maximum transillumination by the endoscope light (fig.1). Palpate the abdominal wall to ensure the colon is not obstructing access to the stomach.
  2. Clean the entire abdominal wall with a disinfectant and put sterile drapes around the puncture site. With the long IM needle, inject a local anaesthetic into the skin and in all layers of the abdominal wall. Insert the IM needle into the cavity of the stomach. Keep the IM needle in place by grasping it with the biopsy forceps. This avoids the stomach wall from dislocating versus the abdominal wall. (fig.2)
  3. With the scalpel, make an incision of  $\pm$  4-5mm in length very close to the IM needle. Push the puncture needle with the trocar through the incision into the stomach cavity (fig.3). Open the biopsy forceps, remove the IM needle and grasp the trocar with the forceps.
  4. Remove the puncture needle and leave the trocar in place. Insert the pull wire through the trocar and grasp the wire with the biopsy forceps. (fig.4)
  5. Dip the dilation tip of the tube in sterile water for approx. 10 seconds to activate the Hydromer® tip-coating. Pass the loop of the wire through the dilation loop, over the internal retention disc and pull snugly. (fig.5)
  6. Pull the wire exiting the abdomen. The tube is then pulled down through the mouth into the stomach cavity and pulled out through the abdominal wall, using the dilation tip to dilate the exit site. The Hydromer® coating on the dilation tip will facilitate this. Remove the trocar when the conical tip of the tube is locked in the intragastric end of the trocar. Pull the internal retention disc of the tube snugly against the stomach wall (fig.6).

- Cut the tube below the conical part, slide the silicone fixation disc over the tube and secure it by placing through the disc. Keep the external fixation disc tension-free against the skin for ca. 24h. Ensure the correct positioning of the external fixation disc is maintained by placing the blue safety clamp immediately behind the disc. Place the quick release clamp over the tube. Submerge the shaft of the feeding connector into water and push the shaft completely into the tube. The tube position should be confirmed endoscopically or by X-ray. Record the brand name, the Charrière, the batch/LOT information and the introduced length of the PEG in to the patients' medical records.

Tube feeding may commence 1 hour after successful PEG placement (ESPEN guideline), however preferably 6 - 8 hours after PEG placement in order to prevent peritonitis.

## MAINTENANCE

The external fixation disc and safety clamp may be loosened 24 hours after placement of the PEG. Allow a distance of approximately 5 mm (fig. 8) between the external fixation disc and the skin, in order to place a sterile Y-compress under the external fixation disc.

Inspect the stoma-site daily for any redness and swelling. Disinfect the skin daily. As soon as the stoma is completely formed, it is sufficient to wash, rinse and dry the skin thoroughly.

Always keep the blue safety clamp in place, also during wound care. The clamp indicates the correct position of the external fixation disc and guarantees that the tube is fixed back in its original position after wound care.

Release the tube out of the external fixation disc, turn the tube daily 180° on its axis, and move the tube up and down in the stoma (min 1.5 cm). Always return the tube in its original position (fig. 9).

Check the position of the Flocare® PEG by confirming the position of the external fixation disc or, if possible, by measuring pH of aspirated gastric content. A pH value below 5.5 confirms correct position of the tube in the stomach. Do this each time you change the feed container, if there is any doubt concerning the position of the tube and at least 3 times a day. (fig.10)

Flush the tube with 20-50 ml of water before and after the administration of feed or medication and at least every 8 hours to prevent tube blockage (fig.11).

- Inspect the condition of the PEG daily. Any significant change to the PEG (discoloration, hairline cracks, leakage, etc.) should be reported to the attending physician.**
- Always keep the blue safety clamp in place.**
- Disinfectants such as Povidone -Iodine (PVP-I; e.g.: Iso-Betadine®, Braunol®) and Octenidindihydrochlorid-Phenoxyethanol (e.g.: Octenisept®) should not be used because repeated exposure to this disinfectant might negatively effect the physical/mechanical properties of the tube. It is recommended to use disinfectants based on Polyhexanid (e.g. Prontosan, Prontosan C.)**

#### Flushing and Syringe use policy

- Use a syringe of at least 20ml for flushing the tube and always flush in a controlled manner.
- For medication delivery, always administer using an appropriate size syringe for the volume of medication but ensure delivery is in a controlled manner.
- Never use excessive force. If there is any resistance, stop the procedure and consult your clinician.

#### DURATION OF USE

The PEG is designed to have a life span of several months, but the life span of the tube varies according several factors. These factors include medication, gastric pH, patient motility and tube care.

Removal of the PEG depends upon the viability of the product and/or should be performed under medical instruction.

- Materials defects and tube occlusion can be largely ruled out if the Flocare® PEG Set is adequately flushed before and after every administration of nutrition and/or medication
- Use liquid medication where possible
- Avoid using acidic substances such as cranberry juice and cola beverages to flush feeding tubes as the acidic quality when combined with formula proteins may actually contribute to tube clogging

#### REMOVAL

The tube should not be removed within 10 days of placement, or until the stoma tract is formed.

The state and position of the tube should be checked endoscopically after 8-10 months.

Removal of this tube depends upon the viability of the product and/or should be performed under medical instruction.

To remove the tube, cut it at skin level. The internal disc and the remaining tube must be retrieved by endoscope.

The Flocare® PEG can be replaced by a replacement gastrostomy (Flocare® Gastrostomy tube).

Oral feeding after PEG removal may only start after medical approval.

**With patients who can no longer be subjected to endoscopy, or endoscopic removal method is not necessary, the PEG tube can be cut off at the skin level and removed, by allowing the internal retention disc to pass naturally. This does result in an increased risk of complications and bowel obstruction. Careful follow-up and monitoring of the patient is essential. This method is contraindicated in children and patients who have previously undergone surgery.**

## SINGLE PATIENT USE

**UK-IE-AU-NZ • Do not reuse this product** - Do not reuse, reprocess or re-sterilise this device. Reuse, reprocessing or re-sterilisation may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or re-sterilisation may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient. After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

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